Gynecology Instrument Tray

Gynäkologie-Gerätekassette

Bandeja para instrumentos ginecológicos

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Gynecology Instrument Tray

Device Description

The Smith & Nephew Gynecology Instrument Tray is designed to contain and protect reusable surgical instruments during transport, sterilization, storage, and to allow optional exposure of the tray's contents to sterilant during the sterilization process.

The tray is a perforated stainless steel case with a latchable lid and handles. The tray is fitted with silicone instrument holders and a protective mat. The tray is marked to facilitate instrument placement.

Indications for Use

The Smith & Nephew instrument tray is intended to contain Smith & Nephew reusable surgical instrumentation for convenient organized storage, sterilization, and transport between uses. The instrument tray must be double wrapped with an FDA-cleared polypropylene sterilization wrap.

The instrument wrap is suitable for use in pre-vacuum steam sterilization method. The instrument tray is not intended to maintain sterility. It is intended to be used in conjunction with a validated sterilization wrap in order to maintain sterility of the enclosed devices. Maintenance of sterility is dependent on the sterilization wrap shelf life and not the tray. Users must follow the instructions for use supplied with the sterilization wrap and follow the manufacturer's recommended storage conditions and time frames for shelf life. The maximum number of devices that could be processed in one load is five. The maximum weight load is 8.4 lbs.

Validated Sterilization Parameters

Method	Cycle
Pre-vacuum steam	Temperature: 132° C (270° F) <p>Exposure time: 4 minutes Drying time: 30 minutes</p>
Instrument Tray Contents	
Device Type	Dimensions
Hysteroscope	Length: 334 mm maximum <p>Diameter: 8 mm maximum Lumen: 3.1 mm minimum</p>
Handpiece	Length: 203 mm maximum <p>Diameter: 32 mm maximum Lumen: 5.1 mm minimum</p>
Sheath	Length: 230 mm maximum <p>Diameter: 9 mm maximum Lumen: 4.6 mm minimum</p>
Calibration insert/obturator	Length: 385 mm maximum <p>Diameter: 4.1 mm maximum Lumen: 1.9 mm minimum</p>
Diagnostic sheath	Length: 390 mm maximum <p>Diameter: 2.9 mm maximum Lumen: 1.9 mm minimum</p>

Device model that is the subject of this pre-market notification:

REF	Description
72203004	Gynecology Instrument Tray

Contraindications

None known.

Warnings

- This product is shipped non-sterile. It must be cleaned before the first use and every subsequent use.
- It is the responsibility of the user to be familiar with the appropriate sterilization technique and the manufacturer's recommended methods prior to use of this device.
- Read these instructions completely prior to use.
- It is the responsibility of the user to perform infection control techniques correctly.
- The instrument tray is not intended to maintain a sterile barrier. The instrument tray must be encased in a validated sterilization wrap during sterilization and storage.
- Inspect the sterilization wrap for breaches prior to use of the instruments. Do not use instruments contained in a tray with a punctured or torn sterilization wrap.
- Do not overload the tray. The effectiveness of the sterilization process may be compromised.

Precautions

- Prior to each use, inspect the device to ensure it is not damaged. Do not use a damaged device.
- Surgical instruments must be thoroughly cleaned and rinsed according to the manufacturer's instructions prior to placement in the instrument tray.
- Do not use the instrument tray with high pH detergents or other caustic chemicals.
- The use of nonabsorbent tray liners such as silicone mats can cause condensate to pool. Adequate drying time for the sterilization cycle should eliminate residual moisture.
- Carefully inspect the wrapped instrument tray following steam sterilization. Repress if moisture is apparent on the outside of the pack.
- The instrument tray must be oriented in the sterilizer to permit adequate steriant penetration. Do not stack the instrument tray directly on top of another one.
- Only legally marketed, FDA-cleared sterilization wrap should be used to wrap the instrument tray.
- Load wrapped instrument tray into the sterilizer or onto the cart or shelf in the upright position with the lid facing up. Do not orient the instrument tray on its side or upside down with the lid facing down.
- Small baskets, trays, or other types of accessories, especially with covers or lids, should be used with the instrument tray only if the tray has been specifically designed and tested for this purpose.
- Use only accessories such as silicone mats, instrument holders, or slotted brackets that come with the instrument tray. These have been designed and validated specifically for use with the Gynecology Instrument Tray.

<div>1</div> <div>Maximum tray load</div>	<div>1</div> <div>Maximalebelastung der Kassette</div>
<div>2</div> <div>Carga máxima de bandeja</div>	<div>2</div> <div>Charge maximale du plateau</div>
<div>3</div> <div>Carga massima vassoio</div>	<div>3</div> <div>Carico massimo vassoio</div>
<div>4</div> <div>Maximal brættelast</div>	<div>4</div> <div>Maximale brættelastning</div>
<div>5</div> <div>Carga máxima do tabuleiro</div>	<div>5</div> <div>Maksimal bakkebelasting</div>
<div>6</div> <div>Maximal brættelast</div>	<div>6</div> <div>Maximal brættelastning</div>
<div>7</div> <div>최대 트레이 적재 용량</div>	<div>7</div> <div>최대 트레이 적재 용량</div>

- Sterilizers vary in design and performance characteristics. It is recommended that the user verify the cycle parameters for the specific sterilizer and type of instruments being sterilized.

Cleaning

▲WARNING: This product is shipped non-sterile. It must be cleaned before the first use and every subsequent use. After each use, the instrument tray and accessories must be cleaned with a neutral pH enzymatic detergent.

Manual Cleaning Procedure

- Remove the tray lid. Remove the pin mat. Remove any shelves or modules. All removable parts should be cleaned independently from the main tray body.
- Soak the pin mat and any removable components in a bath containing an enzymatic, neutral pH detergent (ENZOL® or equivalent) for a minimum of five minutes. Scrub the pin mat and any removable components with a nylon-bristled brush, being careful to access any holes, openings, or maling parts. Use a brush to scrub the pins on the pin mat.
- Rinse the main tray body and the tray lid body with warm running water for not less than 30 seconds. Rotate the tray body and lid to ensure all surfaces are rinsed completely.
- With the tray body and lid in a bath containing an enzymatic, neutral pH detergent (ENZOL® or equivalent), use a nylon-bristled brush to scrub the tray body and lid. Carefully scrub the areas around the holes and the internal instrument holders.
- Rinse the pin mat and any removable components, main tray body, and tray lid thoroughly with warm running or distilled water.

Cleaning Verification

- After cleaning, inspect devices under normal lighting to ensure that all visible soil has been removed.
- If not visibly clean, repeat cleaning and re-inspect.
- For difficult to view areas, 3% hydrogen peroxide may be applied (bubbling is evidence of presence of blood).

Note: Rinse instruments thoroughly with warm tap water following any hydrogen peroxide testing.

The instrument tray and accessories should be inspected following cleaning to verify that there is no damage to the instrument tray, insert tray, or detachable lid, and that the closure system functions properly.

- Discard the instrument tray if there is visible damage or if handles, latches, or instrument brackets become loose.

Instructions for Use

▲WARNING: The instrument tray is not intended to maintain a sterile barrier. The instrument tray must be encased in validated sterilization wrap during sterilization and storage.

▲WARNING: This product is shipped non-sterile. It must be cleaned before the first use and every subsequent use.

CAUTION: Load the wrapped instrument tray into the sterilizer or onto the cart or shelf in the upright position with the lid facing up. Do not orient the instrument tray on its side or upside down with the lid facing down.

CAUTION: Surgical instruments must be thoroughly cleaned and rinsed according to the manufacturer's instructions prior to placement into the instrument tray.

Note: Standard sterility assurance practices for routine monitoring and product testing include the placement of chemical indicators and biological indicators in the most challenging areas of the sterilization load. For the instrument tray, opposite corners at the base, lid and the center of the tray are recommended. For the enclosed instruments, place one instrument with challenging design features and place the chemical or biological indicators on that instrument, in close proximity to the most challenging design features.

- Cover the instrument tray with the lid and secure the lid by completely closing the latches.
- Prior to sterilization, double wrap the loaded tray.
- Sterilize the loaded tray using the validated sterilization cycle listed in "Validated Sterilization Parameters."

Warranty

Smith & Nephew products are guaranteed to be free from defects in material and workmanship for the warranty period for a particular product, beginning from date of invoice. Refer to the current Smith & Nephew Product Catalog or contact Smith & Nephew Customer Service for specific warranty information.

This limited warranty is restricted to replacement by Smith & Nephew, at its option, of any product found to be defective during the warranty period. Damage inflicted to a product by the user that causes it to be unsuitable for refurbishment may result in additional charges, regardless of warranty status. All warranties apply to the original buyer only. In no event shall Smith & Nephew be liable for any anticipated profits, consequential damages, or loss of time incurred by the buyer with the purchase of use of any product.

NO OTHER WARRANTY, EXPRESSED OR IMPLIED, IS GIVEN.

For Further Information

If further information on this product is needed, contact Smith & Nephew Customer Service at 1 800 343 5717 in the U.S., or an authorized representative.

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